# Leeds-Punch

**EN** Instructions for Use

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202-1403 Leeds-Punch

### Description

The Leeds-Punch comprises a single use hand held instrument with a sharp, pointed end and a punch guide / end cap. These are used together to make openings in tissue reinforcement patches to enable sutures to be passed easily through the material.

### Material Specifications

The Leeds-Punch comprises Poca and stainless steel (ASTM F899), which are biocompatible in accordance with ISO 10993-1 and are appropriate for their intended use.

### Intended Use

The Leeds-Punch is intended to create openings in tissue reinforcement patches, by expanding the knitted structure, to facilitate implantation.

### Indications

The Leeds-Punch is indicated for those patients requiring tissue reinforcement patches.

### Contraindications

The Leeds-Punch is only intended for use with the tissue reinforcement patches and should not be used in direct contact with the patient.

### Warnings

- The device is provided sterile and remains so unless the packaging is damaged or opened. Store in standard conditions. DO NOT RESTERILIZE.
- The device is single use only as it is not suitable for reprocessing which amongst other risks may lead to cross-infection and loss of function. Do not use after the expiration date. Discard any open, unused product.
- The surgeon must be thoroughly familiar with these instructions and the recommended surgical procedure before using the device.
- The general principles of patient selection and sound surgical judgement apply.
- The instrument has a sharp, pointed end.

### Precautions

- Inspect the device, packaging and labelling prior to use and do not use if damaged. Contact the

Neoligaments Sales Department if the package has been opened or altered.

- The instrument must not be used by untrained or unqualified personnel.
- The instrument must not be used for any procedure other than the intended use.
- To avoid damaging the instrument do not impact it, subject it to excessive blunt force or use it to manipulate bone.
- Use aseptic technique throughout the procedure.

### Instructions for Use

1. Remove punch guide / end cap from instrument.
2. Position the tissue reinforcement patch so the desired location for an opening is over the hole in the upper face of the punch guide.
3. Push the pointed end of the Leeds-Punch through the patch into the hole in the punch guide.
4. Repeat as necessary for additional openings.
5. Discard all components of the Leeds-Punch instrument.
6. Follow appropriate surgical technique for implantation of the patch.

Further information may be obtained from Neoligaments Sales Department or downloaded from [www.neoligaments.com](http://www.neoligaments.com).

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All rights reserved. © Neoligaments™ 2016.
Neoligaments and Xiros are trademarks of
Xiros Limited, registered in England No. 1664824.